# SHARP

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## Setup Guide

### 1. Download the dedicated [Health Monitor] app.

Please access the app download site via the QR code below.

**[Health Monitor]**  
Download the app

### 2. Start the device.

We recommend charging the device before use.

The device turns on when it is charged.

Place the device on the charging dock and leave it there while charging.

When charging starts, the LED on the charging dock slowly flashes white.

\* For details on charging, see “About Charging” (page 24).

### 3. Follow the instructions on the screen of the [Health Monitor] app on your smartphone to complete the setup.

- Please ensure that Bluetooth® is enabled on your smartphone.
- If the device is not detected, please bring the device and your smartphone closer together and try again.

## ■ Start using

### 1. Attach the main unit

Please wear the ring so that the markings (grooves) on the surface face your palm.

Once data has been accumulated on the device, you can check it on Health Monitor.

Sleep data is recorded by wearing the device while sleeping and is reflected after you wake up.

Place the marker (groove) towards your palm,  
and then attach it to the finger that matches the size.

### 2. Check the device data on Health Monitor

To check the latest data, swipe down (pull the screen downwards) on the Health Monitor home screen.

You can sync data from your device to Health Monitor.

Please operate the app with the device and your smartphone close together.

The device also automatically syncs several times a day.

## About Charging

### ■ About Charging

- 1. Connect a commercially available charging cable (USB Type-C) to the charging dock and a commercially available charging adapter, then plug the adapter into an electrical outlet.**
- 2. Align the indicator (groove) on the surface of the device with the LED position on the charging dock, and place the device on the dock.**  
**When charging starts, the LED on the charging dock slowly flashes white.**
- 3. When the battery is fully charged, the LED on the charging dock changes to a solid blue light.**
  - \* The remaining battery level can be checked in Health Monitor.  
The displayed battery level is an estimate. We recommend charging the device early with sufficient margin.
  - \* If the LED on the charging dock flashes white rapidly, please check that no foreign matter is attached to the device and that the smart ring is properly seated on the charging dock. After confirming this, unplug and reconnect the charging cable (USB Type-C), then try charging again. If the issue still does not improve, please contact the Sharp Wearable Support Center. (See page 40)

- The charging dock uses a USB Type-C connection. Since the connector shape of the charging adapter varies by product, prepare a cable that matches the adapter. Charging using a commercially available charger with a USB Type-C cable is also possible. Do not use products with an unknown manufacturer, distributor, or specifications.
- Do not charge the device using USB ports on computers or smartphones (USB 3.x or USB4 compatible ports).
- The charging dock is not waterproof or dustproof. When charging, be sure to check the following points.
- Check that no water droplets or chemicals (such as skin cream) are attached to the smart ring device. If any foreign matter is present, rinse it off with water, then wipe it dry with a clean, dry cloth and allow it to air-dry completely before charging.
- If any abnormality or damage is found, stop using the product immediately and contact the Sharp Wearable Support Center. (See page 40)

### ■ Regarding not using the product for extended periods

- If you will be storing this product for an extended period without using it, please remove it from the charging cradle and turn off the power from the app screen to reduce battery degradation.
  - \* Tap the connected device in the upper right corner of the screen. You can turn off the power on the next screen that appears.
- Even during long-term storage, charge the device approximately once every six months, then turn it off again before storing it.
- When restarting the device, place it on the charging dock while it is connected to a power source.

- When not using the product for an extended period, wipe off any sweat, dirt, or moisture thoroughly, and store it away from high temperatures, low temperatures, and high humidity.

## Safety Precautions (Be Sure to Follow)

- Before use, please read these "Safety Precautions" carefully and use the product correctly. Also, keep this document in a safe place after reading.
- The precautions listed here are intended to prevent injury to the user or others, and damage to property, so please be sure to follow them.
- The following classifications indicate the severity of potential harm or damage that may occur if the instructions are not followed and the device is used incorrectly.

|                                                                                                  |                                                                                                                          |
|--------------------------------------------------------------------------------------------------|--------------------------------------------------------------------------------------------------------------------------|
|  <b>Danger</b>  | This symbol indicates that "death or serious bodily injury (*1) may directly and immediately result from improper use" . |
|  <b>Warning</b> | This symbol indicates that "death or serious bodily injury (*1) may result from improper use" .                          |
|  <b>Caution</b> | This symbol indicates that "minor injury (*2) and/or property damage (*3) may result from improper use" .                |

\*1 Serious bodily injury: This refers to loss of eyesight, burns (high- and low-temperature), bodily injury, electric shock, broken bones, injuries with residual aftereffects from poisoning, etc., and injuries requiring hospital admission for treatment or long-term hospital attendance as an outpatient.

\*2 Minor injury: This refers to burns (high- and low-temperature), bodily injury, electric shock, etc. that do not require hospital admission for treatment or long-term hospital attendance as an outpatient.

\*3 Property damage: This refers to extended damage to buildings, furniture, livestock, pets, etc.

 **Danger**

**Do not throw into fire or expose to heat.**

- The built-in battery may ignite, rupture, overheat, or leak, potentially causing fire, burns, or injury.

**Do not use, store, or leave the product in places that become hot (such as near a fire, near a heating device, inside a heated table, in direct sunlight, inside a car in direct sunlight, or in a sauna). If you experience heat or discomfort while using the product, remove it from your finger and stop using it until it cools down.**

- This can cause ignition, explosion, fire, burns (high or low temperatures), injury, or malfunction.

**Do not keep the device in or near microwave ovens, induction cookers, or other heating appliances, or pressure cookers or other high-pressure containers.**

- This can cause fire, burns, injuries, electric shock, and malfunctions.

**Do not drill holes or modify the product.**

- This can cause fires, burns, injuries, and other problems.

**The charging base is not waterproof or dustproof. Do not allow water or other liquids (such as drinking water, sweat, seawater, or pet urine) to get into the charging base's connection terminals or allow them to become wet.**

**Do not use the device while it is wet, and do not charge it in places where it may come into contact with water, such as bathrooms or near humidifiers. In addition, do not store the device in places with high temperatures, high humidity, or excessive dust.**

- This can cause fires, burns, injuries, and electric shocks.

**If the charging cradle is completely submerged in water, please refrain from using the device. There may be water remaining inside.**

- This can cause fire, burns, injuries, and electric shock.

 **Warning**

**Do not attach or stick in any metallic objects (stickers, clips, etc. made of metal) on the charging base or the device.**

- This can cause fires, burns, injuries, and other problems.

**Do not cover or wrap the device with cloth or paper while it is charging.**

- This can cause fires, burns, injuries, and electric shocks.

**Do not allow infants or people who are unable to express their own will to use this product.**

**(Be careful not to touch the product)**

- This can cause accidents and other problems.

**To avoid the risk of accidental ingestion, do not place the device within reach of infants or pets.**

- Swallowing the device could cause choking or damage to the stomach and intestines. If you have swallowed it, or suspect you may have swallowed it, seek immediate medical attention and treatment.

**When using the Bluetooth® function, please keep this product at least 15 cm away from the attachment site of medical electrical devices such as implanted cardiac pacemakers and implantable defibrillators.**

- Radio waves can negatively affect the operation of these medical electrical devices.

**For the use of Bluetooth® functionality in hospitals, please follow the instructions of your healthcare provider.**

- This can cause adverse effects on the operation of other medical electrical equipment.

**If you have an implanted cardiac pacemaker or implanted cardioverter-defibrillator (ICD), please consult with your doctor before using the charging dock.**

- Radio waves can negatively affect the operation of medical electrical equipment.

**If any substance from inside the built-in battery leaks, do not allow it to come into contact with your skin (face, hands, etc.) or clothing.**

- This can cause damage to the eyes and skin. If any of the contents get into your eyes or mouth, or come into contact with your skin or clothing, rinse immediately with clean water.

If the product gets into your eyes or mouth, rinse thoroughly and seek immediate medical attention.

**Do not disassemble or modify. (including affixing stickers)**

- This can cause fire, burns, injuries, electric shock, and malfunctions.

**Stop using the device immediately if you notice any abnormalities such as smoke, odor, or noise.**

- Using the device while it is in an abnormal condition may cause fire, burns, injuries, or electric shock.

**When charging or storing the device, always use the supplied charging base.**

- Charging with other devices can cause fire, burns, injuries, electric shock, and damage to the built-in battery, such as leakage, overheating, or explosion.

**If thunder starts to rumble, do not touch the power cable, power adapter, or charging cradle.**

- This can cause electric shock and other dangers.

## Note

**Do not use the product if it is damaged.**

- This can cause fires, burns, injuries, and electric shocks.

**Do not subject the device to strong impacts such as dropping it or placing heavy objects on it.**

- This can cause fire, burns, injuries, electric shock, and malfunctions.

**Do not charge or plug/unplug the power cable when the device is wet with liquid (water, drinking water, sweat, seawater, pet urine, etc.).**

**Also, do not touch the device with wet hands.**

- This can cause fire, burns, injuries, electric shock, and malfunctions.

**Do not soak or wipe with any chemicals such as thinner or benzene.**

- This can cause malfunctions or damage.

**If you notice any abnormal heat generation while charging, stop charging.**

- Charging a device while it is overheating can cause fire, burns, injuries, and other problems.

**Do not store in places with high temperature, high humidity, or excessive dust.**

- This can cause fires, burns, injuries, and electric shocks.

**Make sure the terminals on the charging base are inserted securely.**

- If the plug is not fully inserted, it may cause electric shock or fire due to overheating.

## Handling Precautions

**These are important precautions to prevent product malfunction and to ensure optimal performance. Please read them carefully and use the product properly.**

- This product is not a medical device. This product and Health Monitor are not intended for the diagnosis, treatment, cure, or prevention of diseases or medical conditions, and cannot be used for medical purposes. The information and guidance provided by this product and Health Monitor are intended for reference purposes only and are not a substitute for services such as diagnosis and treatment by a doctor or medical professional. If you have any questions regarding the information or guidance provided by this product or Health Monitor, please consult a medical professional such as a doctor. We will not be held liable for any health problems that may arise as a result of taking action based on the information and guidance provided by this product and Health Monitor. If you make any changes to your sleep or lifestyle based on the information and guidance you receive from this product and Health Monitor, you do so at your own discretion and responsibility, and by doing so, you are deemed to have agreed to the contents of those changes. When using this product and Health Monitor, please carefully observe any changes or reactions in your body and use them within a comfortable range. If you experience unexpected pain, fatigue, or persistent discomfort as a result of changes in your sleep or lifestyle, we recommend that you consult a doctor.

- If you experience redness or skin irritation on your fingers after using this product, discontinue use immediately. If symptoms persist for more than 2 to 3 days without using the device, consult a doctor.
- The accuracy of the device's sensor may decrease depending on the surrounding environment, measurement conditions, and the physiological state of the person wearing it.
- If your heart rate is extremely high or low, it may indicate that the measurement is not accurate.
- If the sensor on the device is dirty, please clean it and try again. If foreign objects prevent the sensor's light from reaching the sensor uniformly, accurate measurements may not be possible.
- The measurement function may not work correctly if the sensor's light is blocked, or due to factors such as skin brightness, blood flow in the skin, or dirt on the sensor.
- While the device is attached, it may get scratched during everyday activities.
- When using exercise equipment, especially free weights such as barbells and dumbbells, or when lifting heavy objects such as metal, ceramic, or stone, please avoid wearing the device as much as possible.
- To avoid damaging the ring, please avoid wearing it with other rings or jewelry made of metal, ceramic, stone, diamonds, or similar materials. If soft metal smartphone covers or jewelry made of gold, silver, aluminum, or other materials are placed in close contact with the device, they may be scratched. Even ceramic smartphone covers can get scratched if they have a soft coating.

- If you experience any skin irritation or discomfort while wearing the device, remove it immediately and consult a doctor.
- Wearing the device for extended periods while wet may cause skin irritation.
- There is a limit to the amount of data that can be stored on the device. If the accumulated data exceeds the capacity of the memory area, older data will be deleted. The device can retain data for approximately four days; however, if the device's battery runs out, all stored data will be erased. To prevent data loss, please sync the device with the app frequently.
- Do not use, store, or leave the product in places that become hot (such as near a fire, near a heating device, inside a heated table, in direct sunlight, inside a car in direct sunlight, or in a sauna).  
If you experience heat or discomfort while using the product, remove it from your finger and stop using it until it cools down.
  - This can cause ignition, explosion, fire, burns (high or low temperatures), injury, or malfunction.
- This product uses green, red, and near-infrared light-emitting diodes (LEDs), but the light intensity is very low, making it safe for both the user and those around them.
- While the device is attached, please avoid handling the battery itself or performing any tasks that involve directly handling the battery. If both the positive and negative terminals of another battery come into contact with the main unit, there is a risk of a short circuit, similar to a regular metal ring. Please take appropriate precautions to avoid this from happening.

## Waterproof/dustproof performance

**The main unit is IPX8\*<sup>1</sup>Equivalent water resistance, IP6X\*<sup>2</sup>It has considerable dustproof performance. The charging base is not waterproof, so we recommend using it indoors. Do not use or store the charging base in high-humidity environments, such as bathrooms.**

- \*1 IPX8 means that when the device is gently submerged in fresh tap water at room temperature to a depth of 1.5 m or more in still water and then removed after approximately 30 minutes, it retains its functionality as a smart ring. This product has been tested under conditions equivalent to a water depth of 100 m.
- \*2 IP6X refers to a degree of protection, meaning that even when held for 8 hours under a maximum reduced pressure of 2kPa inside a device containing dust particles with a diameter of 75  $\mu$  m or less, no dust will enter the device.

## How to Remove the Device

Finger size can change depending on your physical condition and the time of day, which may make it difficult to remove the device from your finger. If you are unable to remove the device, try the following steps. If it still cannot be removed, seek medical attention immediately.

- Lather up some hand soap or bar soap, apply the foam between the device and your finger, and gently remove it by moving it little by little.
- Raise your hand above your heart to temporarily relieve swelling in your fingers, then remove the device while keeping your hand raised.

- In the unlikely event that the device needs to be cut, cutting the device is extremely dangerous. Be sure that the procedure is performed only by a doctor or emergency personnel. When cutting the device, cut only at the indicator (groove) on the outer side of the device. A battery is built into areas other than the indicator (groove), and cutting those areas may result in electric shock or fire.

### About the battery

The types of batteries in the device are as follows:

Display: Li-ion00 Battery type: Lithium-ion battery

### Requests for Disposal

When disposing of the device and charging base, follow the disposal rules of your local government. Rules vary depending on the local government, so please check with the local government of the region you live in.

The lithium-ion batteries used in this device are a valuable resource that can be recycled. Please cooperate with recycling.

Before disposing of the device, please perform a factory reset to restore it to its default settings.

For instructions on how to initialize (reset) the device, please visit the following website.

<https://jp.sharp/support/wearable/>

## **Requests regarding the use of Bluetooth® functionality**

- **The device supports security features compliant with the Bluetooth® standard for communication using Bluetooth® functionality; however, security may not be sufficient depending on the settings. Please exercise caution when using Bluetooth® functionality for communication.**
- **Please be aware that we cannot be held responsible for any data or information leaks that may occur during communication using the Bluetooth® function.**

### **About Wireless Functions**

The device has been certified as wireless equipment for low-power data communication systems under the Radio Law, and as proof of this, the Technical Conformity Mark is printed on the individual packaging box. Please refer to page 60 for the construction design certification number. No radio station license is required to use the device. However, the following actions may be punishable by law:

- Disassembling or modifying the device

### **About the Frequency Bands**

The device can use the 2.4 GHz band from 2.402 GHz to 2.480 GHz, but other wireless devices may also use the same frequencies. To prevent radio wave interference with other wireless devices, observe the following precautions when using the device.

### **Precautions for using Bluetooth® devices**

The frequency band used by the device is also used by home appliances such as microwave ovens, industrial, scientific, and medical devices, as well as licensed indoor radio stations for mobile unit identification used on factory production lines, specified low-power radio stations that do not require licenses, and amateur radio stations (hereinafter, referred to as "other radio stations").

1. Before using the device, please ensure that no other radio stations are operating nearby.
2. In the unlikely event that radio interference occurs between the device and "other radio stations," please immediately change your location or "turn off the power" to avoid the interference.
3. For any other questions, please contact us via the following website.  
<https://jp.sharp/support/wearable/>

**2.4 XX8**

This wireless device uses the 2.4 GHz band. Other modulation methods are used, and the interference distance is 80 m.

### **Body Material List**

| Usage location | Material used   | Surface treatment |
|----------------|-----------------|-------------------|
| Outer          | Titanium        | PVD treatment     |
| Inner          | Synthetic resin | None              |

## Main Specifications

|                                       |                                                                              |
|---------------------------------------|------------------------------------------------------------------------------|
| Body size                             | Thickness 2.8 mm, Width 6.7 mm                                               |
| Body weight                           | 2.1–3.1 g*                                                                   |
| Built-in battery                      | Lithium-ion battery                                                          |
| Built-in battery capacity             | 17–22.5 mAh*                                                                 |
| Bluetooth® Core Specification         | Qualified against Bluetooth® Core 5.0                                        |
| Body Water and Dust Resistance        | IPX8 (equivalent to a water depth of 100 m) / IP6X                           |
| Body Operating Temperature            | –5° C to 45° C                                                               |
| Charging Temperature / Humidity Range | Temperature 5° C to 30° C / Humidity 35% to 80% (Non-condensing environment) |

\* Varies depending on ring size. Ring size availability: 10 sizes.

For the latest and detailed specifications, details are available on the product information website on the back cover.

Please note that specifications and appearance are subject to change without prior notice for improvement purposes.

## Care and Maintenance

- To ensure safer and more comfortable use of this product, please always keep the device software up to date.
- To avoid skin irritation, keep the device clean at all times and wear it correctly on your finger. Leaving it dirty can cause a rash. If skin irritation occurs, discontinue use immediately and consult a doctor.
- If foreign objects are attached to the sensor part of the device, it may affect the detection of the biometric authentication signal from your finger. Therefore, clean the sensor at least once a week. For cleaning, wipe with a soft cloth or hand-wash using neutral detergent and water, then dry thoroughly.
- Depending on the ingredients, sunscreen, lotion, oil, etc. that adhere to the main unit may cause discoloration or rust. When removing dirt, rinse with fresh water and dry thoroughly.

## About Warranty

For customers who purchase this product new, we guarantee a free replacement of the main unit and charging cradle within the scope of the terms and conditions described in the [Free Replacement Policy]. This warranty is valid for one year from the date of purchase.

If a malfunction occurs during the warranty period, contact us at the address below. Our designated courier service will come to your location to pick up the item, confirm the malfunction, and then deliver a replacement.

When receiving warranty service, we ask that you provide proof of purchase (such as the purchase receipt, a copy of the email, or payment details). Please keep this manual and your proof of purchase in a safe place.

★ This does not limit your legal rights.

★ For details regarding the replacement period after the warranty expires or the availability period for repair parts, please contact the Sharp Wearable Support Center below.

**Contact: Sharp Wearable Support Center**

**TEL: 050-3385-8316**

**Reception hours      Mon-Fri(Excluding Saturdays, Sundays,  
national holidays, and Sharp holidays)**

**10:00 ~ 17:00**

**Manufacturer: SHARP CORPORATION**

★ For details of the warranty stipulations, please refer to page 19 (Japanese).

The English version of the warranty stipulations is available on the support page.

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The main unit and accessories may be subject to Japanese export control regulations ("Foreign Exchange and Foreign Trade Act" and related laws and regulations).

Furthermore, it may be subject to U.S. Export Administration Regulations. If you export or re-export the main unit and accessories, you are responsible for taking all necessary procedures and bearing all associated costs. For detailed procedures, please contact the Ministry of Economy, Trade and Industry or the U.S. Department of Commerce.

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### ■ Trademarks

- Company names and product names appearing in this instruction manual are trademarks or registered trademarks of their respective companies.

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Please check the above for details regarding the use of this product, Health Monitor.

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