SHARP

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Introduction

- Please read the Instruction Manual carefully, and use the product correctly and safely.
- Please keep this Instruction Manual in a safe place.
- In this manual, "MH-W01" will be referred to as "this product."
- The initial setup may take some time.
- The illustrations shown in this manual are for illustrative purposes only. The actual product may differ.
- The contents of this manual and the website URLs are subject to change without prior notice.
- Unauthorized reproduction of the contents of this manual, in whole or in part, is prohibited.

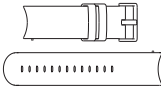
Device Accessories

If any of the items are missing, please contact your retailer.

■ MH-W01 Device X 1

■ Watch Band X 1

■ Charging base X 1

■ Instruction Manual X 1

* This product does not come with a charging cable or charging adapter.

Setup Instructions

- 1. Press and hold the crown (power button) to turn on the device.**
- 2. Download the dedicated [Health Monitor] app.**

Follow the instructions displayed on the device screen, or access the app download site using the QR code shown on the right.

- 3. Follow the on-screen instructions in the [Health Monitor] app on your smartphone to complete the setup.**

- Make sure that your smartphone's Bluetooth® is enabled.
- If the device is not detected, move the device and smartphone closer together and try again.

[Health Monitor]
Download the App

How to Turn the Power On/Off

We recommend charging the device before use. Please refer to page 54 for charging instructions.

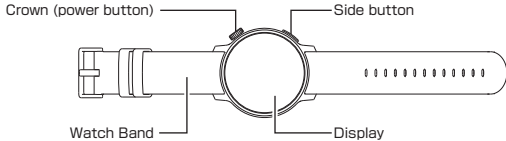
Turn on the device

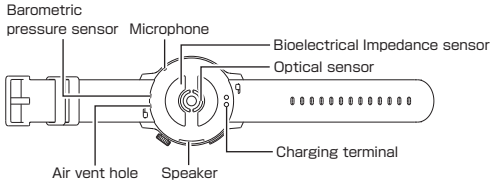
1. Press and hold (for about 7 seconds) the crown (power button) until the screen lights up.

Turn off the device

1. Press and hold (for about 3 seconds) the crown (power button) until the power menu is displayed.
2. Touch [OFF] () on the power menu.
 - You can also turn off the power from [Settings] ⇒ [General] ⇒ [Power off].
 - The time may be inaccurate at startup. By synchronizing with your smartphone, the time will be automatically set to the correct time.
 - Pressing and holding (for about 16 seconds) the crown (power button) will force the device to shut down.

Names of Each Part

About Charging

The charging base is not waterproof or dustproof. Be sure to check the following points during and after charging.

- Make sure that there are no water droplets or foreign matter on this product. If any foreign matter is found, rinse it off immediately. Also, if the device is wet or has been exposed to water, wipe it with a dry, clean cloth and allow it to air dry before charging.

■ Charging

1. Connect a commercially available charging cable (USB Type-C) to the Charging base and a commercially available charging adapter, then plug it into an outlet.

Align the charging terminals and place the device onto the Charging base.

- The terminal on the Charging base is USB Type-C. Since the terminal shape of the charging adapter varies depending on the product, please prepare a compatible charging cable. You can also charge the device using a commercially available charger equipped with a USB Type-C charging cable. Do not use products whose manufacturer, seller, or specifications are unknown.

■ If you do not use the product for a long period of time

- If you are storing this product for an extended period of time and will not be using it, turn off the power.
- If this product is stored without being used for a long period of time, the built-in battery will become over-discharged and the product will no longer be able to be charged or started up. Charge the battery approximately once every six months.
- Storing the battery fully charged for a long period of time will cause it to deteriorate faster. To prolong the life of the built-in battery, do not store it in a fully charged state (immediately after charging is complete).
The recommended battery level for storage is around 50 to 80%.
- If you will not be using the product for an extended period of time, thoroughly wipe off any sweat, dirt, moisture, etc., and store it in a place that is not subject to high temperatures, low temperatures, or high humidity.

Attaching/removing the watch band

Replacing the watch band

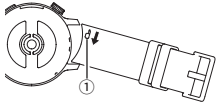
It can be replaced with any compatible watch band up to 20 mm wide.

Removal

1. Slide the spring bar pin (1) on the watch band in the direction of the arrow to remove the watch band from the device.

Installation

1. Insert the opposite end of the spring bar pin of the new watch band into the pin hole on the device first.
2. With pin (1) slid, align it with the pin hole on the device and release the slide. Make sure that the pins on both sides are properly inserted straight into the holes on the device.
3. Replace the other band in the same way.

About wearing

- Measurement accuracy may decrease if the device is not in contact with your skin. Make sure the Bioelectrical Impedance/Optical sensor is in firm contact with your skin when wearing the watch. Do not tighten the band too much, as it may cause discomfort.
- Please securely wear the watch 1 to 2 cm above the wrist bone.
- It is recommended to charge the device after waking up and before breakfast.

About Using This Product

- Before use, please read these "Safety Precautions" carefully and use the product correctly.
- Do not connect this product to a medical device. Also, do not use it together with a medical device.
- In general, measuring your heart rate is recommended as a guideline to determine exercise intensity.
- Exercise according to your physical condition and fitness level, and do not overexert yourself. Sudden or excessive exercise is dangerous. If you experience any changes in your physical condition, such as feeling unwell while exercising, stop exercising immediately and consult a doctor.

- This is not a medical product. This product is designed to record and manage light exercise such as jogging. This product and Health Monitor are not intended to diagnose, treat, cure, or prevent any disease or medical condition and cannot be used for any medical purposes. The information and guidance provided by this product and Health Monitor is for informational purposes only and is not a substitute for the services of a medical professional or physician. If you have any questions about the information or guidance provided by this product or Health Monitor, always consult your medical professional or physician.

We assume no responsibility whatsoever for any health problems that may arise as a result of acting on the information or guidance provided by this product or Health Monitor. If you decide to change your sleep or lifestyle habits based on the information and guidance you receive from this product and Health Monitor, you do so at your own discretion and responsibility, and you will be deemed to have agreed to the changes.

When using this product and Health Monitor, carefully observe any changes or reactions in your body, and use them within your limits. For example, if you experience unexpected pain, fatigue, or prolonged discomfort, or other symptoms after making changes to your sleep or lifestyle habits, we recommend that you consult your doctor.

- To accurately measure daily calorie intake and calorie expenditure, it is important to wear the device for 22–23 hours per day, including during meals and sleep.
- Regularly updating your body weight data (2–3 times per week) helps improve the accuracy of calorie calculations.
- If your heart rate is extremely high or low, it may not have been measured accurately.
- Please note that calorie intake may not be measured accurately during activities such as bathing or showering.

Safety Precautions (Be Sure to Follow)

- **Before use, please read these "Safety Precautions" carefully and use the product correctly. Also, keep this document in a safe place after reading.**
- **The precautions listed here are intended to prevent injury to the user or others, and damage to property, so please be sure to follow them.**

- The following classifications indicate the severity of potential harm or damage that may occur if the instructions are not followed and the device is used incorrectly.

 Danger	This label indicates that improper handling may result in "imminent danger of death or serious injury (*1)."
 Warning	This label indicates that improper handling may result in "death or serious injury (*1)."
 Caution	This label indicates that improper handling may result in "minor injury (*2) and/or property damage (*3)."

- *1 Serious injury: Refers to injuries that leave permanent damage, such as blindness, burns (high or low temperature), injuries, electric shock, fractures, or poisoning, as well as injuries that require hospitalization or long-term outpatient treatment.

- *2 Minor injury:Refers to burns (high/low temperature), injuries, electric shock, etc., that do not require hospitalization or long-term outpatient treatment.
- *3 Property damage:This refers to extensive damage to houses, household goods, livestock, pets, etc.

Handling instructions

Do not use, store, or leave the product in places where it may become hot (near a fire, near heating equipment, inside a heated table or kotatsu, in direct sunlight, inside a car under the hot sun, in a sauna, etc.).

If you feel heat or discomfort while using the device, remove it from your wrist and stop using it until it cools down.

- This may cause ignition, explosion, fire, burns (high- or low-temperature), injury, malfunction, etc.

Do not throw into fire or overheat the device.

- This may cause ignition, explosion, fire, burns, injury, etc.

Do not place the device in heating appliances such as a microwave oven, or in high-pressure containers.

- This may cause ignition, burns, injury, electric shock, etc.

Do not subject the device to strong force, impact, or vibration such as by driving a nail into it, hitting it with a hammer, placing a heavy object on it, dropping it, stepping on it, or throwing it.

- This can cause built-in battery ignition, explosion, heat generation, leakage, fire, burns, and electric shock.

If the charging base has been completely submerged in water, do not use it. Water may remain inside.

- This may cause fire, burns, injury, electric shock.

Do not allow the charging base to become wet with water or other liquids (such as drinking water, sweat, seawater, or pet urine). Do not use the device when it is wet.

- This may cause fire, burns, injury, electric shock.

Also, do not charge the device when it is wet, or in places where it may come into contact with water such as bathrooms or near humidifiers.

- This may cause fire, burns, injury, electric shock.

Do not disassemble or modify the product.

- This may cause fire, burns, injury, electric shock.

Precautions regarding magnetic influence

This product (including the charging base) contains magnets. Please observe the following precautions.

- It may affect medical devices such as implantable cardiac pacemakers and programmable pressure shunts used to treat hydrocephalus.
If you use any of these medical devices, keep this product away from you.
If you are using any medical devices, be sure to consult your doctor before using this product.
- Do not place items that are affected by magnets, such as credit cards, debit cards, bankbooks, subway or train magnetic tickets, or parking tickets, near this product.
Magnetic forces may erase the information or make them unusable.

If any substance inside this product gets into your eyes or mouth, etc., immediately rinse with clean water and seek medical attention.

- The substances inside this product may cause blindness or poor health.

When charging the device, always use the supplied charging base.

- Use of any product other than the specified ones may result in heat generation, ignition, explosion, or leakage of the built-in battery.

Use only genuine software.

- If software other than genuine software is used, it may cause fire, burns, injury, electric shock, etc.

Warning

Do not allow conductive foreign objects (such as metal pieces or pencil lead) to come into contact with the charging terminals or the connection terminals of the charging base, or allow dust or sand, etc., to get inside.

- This may cause fire, burns, injury, electric shock, etc.

Do not cover or wrap the device with bedding or other materials while charging.

- This may cause fire, burns, etc.

If you start to hear thunder, do not touch the charging base.

- This may cause an electric shock.

Do not short-circuit the charging terminal while it is connected to a power outlet. Also, do not allow your hands, fingers, or any part of your body to touch the charging terminal.

- This may cause fire, burns, electric shock, etc.

If you notice any abnormalities such as strange odors, strange noises, smoke, heat generation, discoloration, or deformation, etc., during use or storage, take safety precautions, turn off the power, and do not use the product.

- This may cause fire, burns, injury, electric shock, etc.

Please be careful not to let pets bite this product.

- The damage may cause fire, burns, injury, etc.

For use inside gas stations and other facilities, please follow the instructions of each facility.

- When using the product inside a gas station, etc., be careful not to drop it, and do not use it while refueling.

Doing so may ignite flammable gases, causing an explosion or fire, etc.

Caution

Do not use the product if it is damaged.

- This may cause fire, burns, injury, etc.

Do not place the device within reach of infants or pets.

- This may cause accidental ingestion, injury, or electric shock.

Do not use or store the product in areas with high humidity, excessive dust, or high temperatures.

- This may cause fire, burns, electric shock, etc.

For information on waterproof and dustproof performance, please see the following.

⇒ P.85 "Waterproof/Dustproof Performance"

If the substance inside this product leaks, do not let it come into contact with your skin (face, hands, etc.) or clothing.

- It may cause injury to the eyes or skin. If any of the internal substances gets into your eyes or mouth, or on your skin or clothing, rinse immediately with clean water. If any of the internal substances gets into your eyes or mouth, rinse immediately and seek medical attention.

Make sure the terminals on the charging base are inserted securely.

- If the plug is not inserted completely, it may cause electric shock or fire due to heat generation.

Do not operate the button while it is wet.

- Water may get inside and cause the waterproofing to become impaired.

If you experience any skin abnormalities while using this product, stop using it immediately and consult a doctor.

- Depending on your constitution and physical condition, you may experience itching, rashes, eczema, or other symptoms.
⇒ P.83 "Material list"

The device and charging base contain magnetic components, so make sure there are no metal pieces (such as cutter blades or staples) attached to them.

- The attached objects may cause injury, etc.

Handling near medical electrical equipment

Warning

When using the Bluetooth® function, keep this product at least 15 cm away from the implantation site of medical electrical equipment such as implantable cardiac pacemakers and implantable defibrillators.

- Radio waves can cause adverse effects on the operation of such medical electrical equipment.

To use the Bluetooth® function in a hospital, please follow the instructions of the medical institution.

- It may cause adverse effects on the operation of other medical electrical equipment.

If you are using an implanted cardiac pacemaker, implanted cardioverter defibrillator, or other medical electrical device, please consult your doctor before using the charging base.

- Radio waves can cause adverse effects on the operation of medical electrical devices.

Handling Precautions

■ Common requests

- These are important precautions to prevent product malfunction and to ensure optimal performance. Please read them carefully and use the product properly.
- The measurement function may not work properly depending on factors such as the sensor light being blocked, the surrounding conditions, the measurement conditions, the physiological state of the wearer, or dirt on the sensor.
- Avoid extreme high or low temperatures.
Use within the temperature range of 5°C to 35°C, and humidity range of 35% to 85%.
Please charge the device within the same temperature and humidity range.

- Sudden temperature changes can cause condensation.
When bringing this product from a cold place into a warm bathroom, etc., wait until the product has reached room temperature before bringing it in.
Do not use the product until the condensation has evaporated, as this may cause a malfunction.
- Do not place near an air conditioner vent.
Sudden temperature changes can cause condensation, which can lead to internal corrosion and malfunction.
- Although the device is waterproof, prolonged use while wet may have an adverse effect on the skin. Allow to dry thoroughly before use.
- Remove this product from your arm at regular intervals to allow air to reach your skin.

■ About the Built-in Battery

The type of built-in battery of the device is as follows:

Display: Li-ion00 Battery type: Lithium-ion battery

■ Requests for Disposal

When disposing of this product, follow the disposal rules of your local government. Rules vary depending on the local government, so please check with the local government of the region you live in.

The lithium-ion rechargeable battery used in the device is a recyclable and useful resource. Please cooperate with recycling.

Before disposing of the device, please reset it to its factory settings.

[Settings] ⇒ [General] ⇒ [Reset]

■ Requests when Using the Bluetooth® Function

Note that we cannot be held responsible for any damages arising from data or information leakage during use of the Bluetooth® function. Use the Bluetooth® function of this product only within Japan. The Bluetooth® function of this product is certified for wireless use only within Japan. Use outside Japan may be subject to penalties.

About Wireless Functions

This product has been certified as wireless equipment for low-power data communication systems based on the Radio Act, and as proof of this, the "Technical Conformity Mark  " is displayed on the electronic nameplate of the device. The electronic nameplate on the device can be checked by following the steps.

[Settings] ⇒ [About watch] ⇒ [Regulatory information]

No radio station license is required to use this product. However, the following actions may be punishable by law:

- Disassembling or modifying this product

About the Frequency Bands

This product can use the 2.4 GHz band from 2.402 GHz to 2.480 GHz, but other wireless devices may also use the same frequencies. To prevent radio wave interference with other wireless devices, observe the following precautions when using the device.

Precautions when Using Bluetooth® Devices

The frequency band used by this product is also used by home appliances such as microwave ovens, industrial, scientific, and medical devices, as well as licensed indoor radio stations for mobile unit identification used on factory production lines, specified low-power radio stations that do not require licenses, and amateur radio stations (hereinafter, referred to as "other radio stations").

1. Before using this product, make sure that there are no "other radio stations" operating nearby.

2. In the unlikely event that radio wave interference occurs between this product and "other radio stations," immediately change the location where you are using the device or "turn off the power" to avoid the radio wave interference.
3. If you have any other questions, please contact us via the following website.
<https://jp.sharp/support/wearable/>

This wireless device uses the 2.4 GHz band. The modulation method is FH-SS, and the interference distance is 80 m.

■ Voluntary control of radio interference

This product complies with the technical standards based on the Voluntary Control Measures Implementation Rules published by the VCCI, and the conformity mark is displayed on the product's electronic nameplate.

The electronic nameplate on the device can be checked by following the steps.

[Settings] ⇒ [About watch] ⇒ [Regulatory information]

- Do not make unauthorized modifications to the operating software of the device.

We may consider this to be software modification and refuse to repair the device in the event of a malfunction.

■ Material list

Usage location	Material used	Surface treatment
Buckle	Stainless steel	PVD treatment AF Coat
Watch Band	Fluoroelastomer	Soft Touch Coating
Display surface (glass part)	Tempered glass	AF Coat
Side	Stainless steel	PVD treatment AF Coat
Crown (power button)	Stainless steel	PVD treatment AF Coat
Side button (surface)	Stainless steel	PVD treatment

Usage location	Material used	Surface treatment
Side button (inner side surface)	PA resin (glass-filled)	-
Back	PA resin (glass-filled)	Coating
Bioelectrical Impedance sensor	Titanium	-
Optical sensor	PA resin	Coating
Charging terminal	Stainless steel	Nickel plating Gold plating

Waterproof/Dustproof Performance

The device has water resistance equivalent to 5 ATM (*1) and IPX8 (*2), and dustproof performance equivalent to IP6X (*3).

- *1 Water resistance of 5 bar (5 ATM) means that the device can maintain its waterproof performance even when subjected to a pressure of 5 bar underwater for 10 minutes.
- *2 IPX8 means that when the device is gently submerged in still tap water at room temperature to a depth of 1.5 m for approximately 30 minutes, it retains its function as a watch.
- *3 IP6X refers to the level of protection against dust, meaning that even when the device is placed in an environment containing dust particles with a diameter of 75 μ m or less and subjected to a maximum differential pressure of 2 kPa for 2 hours, no dust penetrates the interior.

- The watch is waterproof to 5 atmospheres. It can be used by people who frequently work in wet environments (such as farming, car washing, or food service) or who swim in pools.
- Do not immerse in seawater.
- Ingredients in detergents, soap, bath additives, and hot springs may degrade waterproof gaskets and metal parts. If it becomes soiled, rinse thoroughly with clean water before wiping.
- Do not use this product in a sauna.
- Do not use for skin diving, saturation diving, or scuba diving. Also, be careful not to expose the device to high-temperature or high-pressure shower or tap water.

- The device can be washed with hand soap. It can be washed using foam-type hand soap for household use manufactured in Japan (according to SHARP CORPORATION). Do not use other types of detergents, such as alkaline hand soap, dishwashing detergent, bar soap, or powdered soap. The performance of this device has been verified under test conditions, and it does not guarantee that it can be washed with all types of foam-type hand soap.

Care and Maintenance

- To clean, wipe with a dry, soft cloth. Rubbing too hard may cause scratches.
- Wiping with alcohol, thinner, benzene, detergent, etc., may cause the printing to disappear, the color to fade, or the surface finish to change.
- To avoid skin irritation, always keep this product clean and wear it correctly on your wrist. Leaving it dirty can cause a rash. If skin irritation occurs, discontinue use immediately and consult a doctor.
- If any foreign matter is adhering to the sensor, it may affect the detection of the biometric authentication signal. Therefore, clean the sensor at least once a week. For cleaning, wipe with a soft cloth or hand-wash using neutral detergent and water, then dry thoroughly.
- To remove dust from the device and the terminals on the charging base, periodically wipe them off with a dry cloth or brush when not charging.

System Updates

System updates are a feature that allows you to use this product more safely and comfortably. Please be sure to update to the latest system.

- Place the device on the charging base when the battery level is at least 30% and follow the instructions on the device screen.
- Do not forcibly shutdown the device during a system update.

About Warranty

For customers who purchase a new product, free replacement of the device and charging base is warranted within the scope of the contents described in [Free Replacement Policy]. This warranty is valid for one year from the date of purchase.

If a malfunction occurs during the warranty period, contact us at the contact information on the next page. Sharp's designated delivery company will come to pick up the product, and will deliver a replacement product after checking the fault condition.

When receiving warranty service, we ask that you provide proof of purchase (such as the purchase receipt, a copy of the email, or payment details). Please keep this manual and your proof of purchase in a safe place.

- ★ This does not limit your legal rights as a customer.
- ★ For more information on the replacement period after the expiration of the warranty period or the holding period of repair performance parts, please contact the "Sharp Wearable Support Center" on P.91.

Contact Information: Sharp Wearable Support Center

TEL: 050-3385-8316

Reception hours **Monday to Friday (excluding
Saturdays, Sundays, public holidays,
and Sharp holidays)**

10:00 to 17:00

Manufacturer: SHARP CORPORATION

[Free Replacement Policy]

1. If the hardware of this product breaks down during the warranty period under normal use in accordance with the instructions in the Instruction Manual and on the warning labels on the product, we will collect it free of charge in accordance with this manual and replace it.
 - Failures not caused by the hardware of this product (e.g. Problems caused by software installed by the customer, or malfunctions due to virus infection in the connected devices) are outside the scope of this manual.
2. Even during the warranty period, the replacement would be charged in the following cases:
 - (a) If the proof of purchase (such as the purchase receipt, a copy of the email, or payment details) is not provided

- (b) In the case of failure or damage caused by improper use, improper repair or modification, or any other violation of the prohibitions set forth in the Instruction Manual or similar documents.
- (c) If the product is dropped, or in the case of failure or damage caused by use under conditions that exceed the waterproof performance described in the Instruction Manual, or due to water entering a non-waterproof charging base.
- (d) In the case of failure or damage caused by external factors such as fire, pollution, abnormal voltage, use of a power source (voltage, frequency) outside the rated range, earthquakes, lightning strikes, gusts of wind, floods, salt damage, gas damage (such as sulfur gas), and other natural disasters.
- (e) When replacement is required due to natural wear, abrasion, deterioration, etc. of parts with a limited lifespan or consumables (limited to parts that are likely to wear out within the warranty period if used for longer or more frequently than normal).

- (f) In the case of failure or damage to the product caused by the use of other connected devices or improper consumables or storage media.
 - (g) In the case of failure or damage caused by software provided by a third party other than Sharp.
 - (h) In the case of a failure or damage caused by the customer's usage environment or maintenance/management method (e.g., a failure caused by the intrusion of dust, rust, mold, insects, etc.).
 - (i) Any other failure or damage for which Sharp is not responsible.
3. Even when the product is within the warranty period, if the holding period of the replacement product after discontinuation of the product as separately specified by Sharp has expired, and if the replacement product cannot be procured, we may not be able to replace the product.

4. The damaged product received for replacement will be collected and disposed of appropriately by Sharp.
5. The warranty period of products that have been replaced free of charge based on this document shall be the same as the warranty period at the time of original purchase.
6. Sharp shall not compensate for any damages caused by the product being unusable due to a failure.
7. Sharp shall not be able to provide any support for data recovery or transfer due to product failure.
8. Before requesting for replacement, it is your responsibility to synchronize the data with the application and then reset (Initialization) the product to delete the data. All data received during replacement will be deleted. We cannot compensate for any secondary damage caused by the deletion of product data or other data.
9. This warranty is valid only in Japan.

Main Specifications

Body size	Diameter approximately 42 mm, thickness approximately 9.4 mm (thickest part 11.9 mm)
Body weight	Approximately 33 g
Built-in battery	Lithium-ion battery
Built-in battery capacity	270 mAh
Bluetooth® Core Specification	Qualified against Bluetooth® Core 5.4
Body Water and Dust Resistance	5 atmospheres (5 ATM), equivalent to IPX8 for water resistance, and equivalent to IP6X for dust resistance

Operating temperature/ humidity	5°C to 35°C/35% to 85%
------------------------------------	------------------------

For the latest and detailed specifications, details are available on the product information website on the back cover.

Please note that specifications and appearance are subject to change without prior notice for improvement purposes.

Export Control Regulations

The device and its accessories may be subject to Japanese export control regulations ("Foreign Exchange and Foreign Trade Act" and related laws and regulations). In addition, the United States Export Administration Regulations may apply. When exporting or re-exporting the device or its accessories, it is necessary to take the required procedures at your own responsibility and expense. For detailed procedures, please contact the Ministry of Economy, Trade and Industry or the U.S. Department of Commerce.

Trademarks

- Calorie measurement and hydration measurement uses FLOW™ technology from HEALBE. FLOW™ is the registered trademark by HEALBE.
- QR Code is a registered trademark of DENSO WAVE INCORPORATED.
- Bluetooth® is a registered trademark of Bluetooth SIG, Inc. in the United States.
- Android is a trademark of Google LLC.
- USB Type-C™ is a trademark of USB Implementers Forum.
- SHARP logo, Health Monitor are registered trademarks of SHARP CORPORATION.
- Other company names and product names are trademarks or registered trademarks of their respective companies.

Open Source Software

- This product includes open source software. These software programs are used in accordance with their respective license conditions.
For details regarding the software license, please refer to [≡] ⇒ Select a connected device ⇒ [Device Information] ⇒ [Legal information] ⇒ [Open source Licenses] on Health Monitor.

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